**ENGLISH**  
Oragene<sup>™</sup> DNA Self-Collection Kit  
User Instructions

**Do not eat, drink, smoke or chew gum for 30 minutes before giving your saliva sample.**

- 1 Start giving your saliva sample.
- 2 Spit until the amount of liquid saliva (not bubbles) reaches at least the level shown in picture #2.
- 3 Put the container on a flat surface. Screw the cap onto the container. Make sure that the cap is closed tightly, as shown in picture #3c. If you have trouble closing the cap all the way, turn the cap slightly counter-clockwise then try to close it again.
- 4 Mix gently for at least 10 seconds.  
Recycle outer packaging.

- User Tips:**
- Do NOT remove plastic film from the cap.
  - To make more saliva, close your mouth and wiggle your tongue or rub your cheeks. Some people find it easier to spit more if they place ¼ tsp of plain white sugar on your tongue.
  - Finish spitting within 30 minutes.

**Intended Use:** This product is designed for the safe collection of saliva samples.

**Contents:** Each collection vial contains 2ml of Oragene liquid. Before use, the solution in the cap should be clear and colorless.

**Warnings:** Wash with water if the Oragene liquid comes in contact with eyes or skin. Do not ingest.

**Storage:** Store at room temperature 15-30 °C (59-86 °F).

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